



Adjustable Headrest  
& Chest Pad

Removeable  
Knee Pads

## SETUP PROCESS

The following steps will assist you with the setup of your Ergo Pro II™ portable Vitrectomy chair.

### Unpacking the Chair

1



Remove all items from the box: Ergo Pro II, FacePillow and Deluxe Adjustable Platform. Lay the chair facing up.

### Releasing the Frame

2



Locate the elastic ball-lock under the Stronglite logo on the faceplate. Pull the ball upward.

3



Release the ball-lock away from the faceplate, allowing the chair to be opened.

### Standing the Chair Up

4



Pull the chair up, onto the back legs.

5



Anchor your foot on a back leg (A), near the wheel, & pull the front legs all the way forward till the chair stands upright (B).

### Locking the Frame

6



7



At the front of the chair, locate the frame lock lever underneath the arm rest. Carefully press the lever down completely till it locks in place. **IMPORTANT: This step prevents the metal frame from collapsing while in use.**

### Arm Rest Adjustment

8



Locate arm rest lock lever (A). Lift armrest upward (B).

9



Fold lock lever over & down to secure arm rest.

### Chest Pad Adjustment

10



Locate chest pad lock lever (A). Make sure the lever is loose.

## SETUP PROCESS CONTINUED

### Chest Pad Adjustment (cont'd)



Adjust chest pad to desired location (B).



Fold lock lever over & down to secure chest pad.

### FacePillow & Platform Adjustment



Locate platform lock lever in 'open' position (A). Fold down platform arms to desired location (B). Fold lock lever down to secure arms in place (C).

### FacePillow & Platform Adjustment



Place FacePillow onto platform (A). Insert platform arms into inserts underneath the chest pad (B). Fold lock lever over & down to secure platform (C).



Use circular wheel tighteners for adjusting height level of the platform.

### Setup Complete

Your Ergo Pro II is now ready for work!




## ADDITIONAL FUNCTIONALITY

The following steps will help you customize the Ergo Pro II to fit all your clients.

### Seat Angle Adjustment



The seat pivots up from the back legs. Locate and press the round metal pin inward to release the seat.

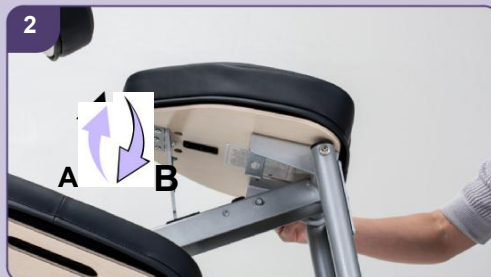

Lift the seat up (A). Pull the seat height lever down into the receiver slot (B).




The receiver location allows the seat to be locked in two angles: high or low.



## ADDITIONAL FUNCTIONALITY

The following steps will help you customize the Ergo Pro II to fit all your clients.

### Chair Height Adjustment - Back Legs

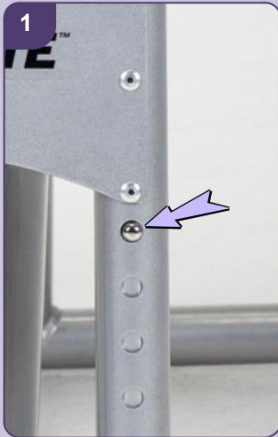

1  
Locate the round metal pins on each back leg under the faceplate.



2  
Press the round metal pins into the legs to allow for height adjustment.



3  
Adjust the legs to the desired height increment.

### Removable Pads



- FacePillow
- Arm Rest
- Chest Pad
- Seat Pad
- Knee Rests

All the pads and cushions for your Ergo Pro II can be removed and replaced!

### Removing the Knee Rests for Traditional Sitting Position



1  
Locate the round metal pins next to each support guide on the knee rests. Press each pin down to loosen the knee rest.



2  
Slide the knee rest forward until the support guide is centered within the circular opening.



Remove the knee rests. Your Ergo Pro II can now be used with a regular sitting position.